



## Wynstan Health Face Mask 4-layer Level 2 Australian Made 50/PK

**\$31.90**

In stock  
**SKU** WH404101

 [Add to Wish List](#)

 [Add to Compare](#)

50 pack - Level 2 rated surgical disposable face mask designed for use in settings where minimal blood/droplet exposure is likely. This includes changing dressings on small or healing wounds in primary care or the emergency department and for use in dentistry procedures.

### Details

50 pack - Level 2 rated surgical disposable face mask designed for use in settings where minimal blood/droplet exposure is likely. This includes changing dressings on small or healing wounds in primary care or the emergency department and for use in dentistry procedures.

### PRODUCT FEATURES

- AS 4381 :2015 Level 2 rating for:
- Differential pressure (< 5.0 mm Hp! cm2 )
- Resistance to penetration by synthetic blood (120 mmHg)
- Bacterial filtration efficiency (98%)
- High quality design and construction
- Comfortable four-layer filtration with good breathability
- Wide and firm adjustable nose bridge to reduce fogging
- One size fits all
- Latex free
- Proudly Australian designed, manufactured and owned

[Fitting Instructions](#)

### FITTING INSTRUCTIONS

1. Place mask across the bridge of your nose and attach ear loops. Ensure the mask covers your nose, mouth and chin
2. Press the nose clip comfortably and evenly across the bridge of the nose to ensure a good seal



3. Press the mask firmly onto the face
4. Gently pull down the base of the mask until you create a close fit to both sides of your face
5. Follow appropriate Health Care Institution surgical face mask "fit & check" protocols
6. This face mask can be used continuously for up to 4 hours, as long as the mask does not become moist, soiled or damaged
7. Where single-use face masks are used for extended periods and are needed to be removed to eat and/or drink, hand hygiene and sanitizing should be maintained
8. This face mask must not be stored or reused after removal

## **PRODUCT CONSTRUCTION**

- Outer layer: 100% Spunbon Polypropylene
- Middle layer 1: 100% Melt Brown Polypropylene
- Middle layer 2: 100% Spunbond Polypropylene
- Inner layer: 100% Spunbond Polypropylene

## **PRODUCT STORAGE**

Product should be stored in a clean dry environment away from direct sun light under 30 degrees Celsius