



Clinell Universal Wipes Packet 200



\$24.15

In stock
Only **94** left
SKU CW200AUS

 [Add to Wish List](#)

 [Add to Compare](#)

Clinell Universal Wipes (pack of 200) are hospital-grade surface disinfectant wipes proven effective against bacteria, viruses, fungi and spores in 1 minute. Used across the NHS, Australian hospitals, GP clinics and dental surgeries for daily infection control on surfaces, medical equipment and high-touch areas.



Details

What are Clinell Universal Wipes?

Clinell Universal Wipes are the gold-standard one-step disinfectant wipes used across UK and Australian hospitals, clinics and dental surgeries for daily infection control. The proprietary biocidal formula kills 99.999% of pathogens - including bacteria, viruses (enveloped and non-enveloped), fungi, spores and tubercle bacilli - within 1 minute of contact. The 200-wipe pack is designed for clinical and high-use settings.

What Clinell wipes kill

- **Bacteria** - including MRSA, E. coli, Salmonella, Pseudomonas, VRE, ESBL.
- **Viruses** - including coronaviruses, influenza, norovirus, rotavirus, HIV, hepatitis B and C.
- **Fungi** - including Candida albicans.
- **Spores** - including Clostridium difficile.
- **Mycobacteria** - including Mycobacterium tuberculosis.



Key features

- One-step cleaning + disinfection - no pre-clean required.
- 1-minute contact time for full kill claim.
- Alcohol-free - safe on most medical equipment including screens.
- Non-corrosive, non-irritant formula.
- Compatible with stainless steel, plastics, vinyl, painted surfaces, glass.
- Detergent action lifts soil and biofilm.
- Compliant with EN 13727, EN 13624, EN 14476, EN 13704 European disinfection standards.

Where Clinell wipes are used

- Hospital wards and operating theatres.
- GP and specialist clinic surfaces, treatment chairs and equipment.
- Dental surgery dental units, light handles and tray tables.
- Ambulance and emergency vehicle decontamination.
- Aged care facilities and rehabilitation centres.
- Childcare centres and schools for daily surface disinfection.
- Veterinary clinics and animal-treatment surfaces.

How to use Clinell Universal Wipes

Open the pack and pull out a wipe through the resealable lid. Wipe the surface thoroughly, ensuring complete wet coverage. Allow the surface to stay visibly wet for at least 1 minute for full disinfectant claim. Dispose of the used wipe in general waste (do not flush). Reseal the pack lid to prevent the remaining wipes drying out.

For heavily soiled surfaces, wipe once to clean, then use a second wipe to disinfect.

Warnings and precautions

- For surface use only - not intended for skin disinfection.
- Keep pack lid sealed between uses to maintain wipe saturation.
- Do not flush wipes - dispose in general waste.
- Avoid contact with eyes. If contact occurs, rinse with water.
- Store below 30 degrees Celsius away from direct sunlight.

Frequently asked questions

What pathogens do Clinell Universal Wipes kill?

Clinell Universal Wipes kill 99.999% of bacteria (including MRSA, E. coli, VRE, ESBL), enveloped and non-enveloped viruses (including coronaviruses, norovirus, HIV, hepatitis B/C), fungi, spores (C. difficile) and mycobacteria within 1 minute.



Are Clinell wipes alcohol-free?

Yes. Clinell Universal Wipes are alcohol-free, making them safe on most medical equipment, electronic touchscreens, plastics and surfaces that alcohol can damage. This is one of the reasons they are widely chosen in clinical settings.

How long do I need to keep a surface wet with Clinell wipes?

The surface needs to stay visibly wet for at least 1 minute after wiping for the full bacterial, viral and fungal kill claim. For *C. difficile* spores, the contact time is 5 minutes.

Can I use Clinell Universal Wipes on skin?

No. Clinell Universal Wipes are surface disinfectant wipes only. For skin antisepsis use 70% alcohol wipes (Medi-Swabs) or a chlorhexidine skin preparation.