



MOOV Head Lice Comb

£5.84

In stock
Only **200** left
SKU 11550

 [Add to Wish List](#)

 [Add to Compare](#)

Helps to easily detect and remove head lice and eggs.



Details

Helps to easily detect and remove head lice and eggs.

MOOV Head Lice Removal Comb helps to easily detect and remove head lice and eggs. Built with strong steel and precisely aligned teeth, the MOOV Head Lice Removal Comb helps catch lice and eggs while the rounded tips and angled handle make it comfortable to hold and gentle on scalps.



Directions

1. Saturate hair - after treatment apply conditioner.
2. Massage product into hair, particularly near scalp, ensuring hair is saturated.
3. Start combing - divide hair into segments. Starting at the scalp, comb out lice and eggs with comb. Wipe combings onto a tissue as you proceed.
4. Continue combing until no eggs or lice are found. Wash or rinse hair thoroughly. Rinse comb clean with warm water.

Cautions

Keep out of reach of children.